

# clinell®

## Detergent Range

---

Stronger cleaning  
power, ultra-low smear



# Detergent Wipes

Cutting edge technology combined with ease of use to create an incredibly effective and environmentally friendly formula.

**Patented formula**

Safe and biodegradable formulation for an ultra-low smear clean.

**Optimum cleaning**

Even stronger cleaning power, developed specifically for use in healthcare environments.

**Convenient and effective**

Disposable wipes that are ideal for general cleaning and damp-dusting.

**Multi-surface cleaning**

Can be used on both surfaces and equipment.

**Range of uses**

(Examples of use - this is not an exhaustive list).

**Surfaces**

- Mattresses
- Bed frames
- Table tops
- Desks
- Chairs
- Ledges
- Examination couches

**Equipment**

- Telephones
- Keyboards
- Accessories
- Wheelchairs
- Trolleys
- Walking frames

# Optimum cleaning

Multi-surface detergent wipes for general cleaning and damp dusting that provide a superior level of cleaning.

Clinell detergent wipes are unlike any other. Our unique formula is incredibly effective at providing optimum multi-surface cleaning and damp dusting within a healthcare setting. Many other detergent products are oil and phosphate based, which leaves smears and residue.

Our patented formula use compounds that lower the surface tension of a liquid and act as a detergent. The sufficient molecules are designed for ultra-low smearing and excellent cleaning power.

Containing no alcohol or disinfectant, Clinell Detergent Wipes are extremely safe to use and are also environmentally friendly.

## How to use Clinell Detergent Wipes



Wash or decontaminate your hands and put on PPE (risk assessment to ensure appropriate PPE is worn, as per local protocol).



Remove one wipe from the pack.



Working from top to bottom, clean to dirty, wipe in an 'S' shaped pattern.



Should your wipe dry out or become soiled, replace it with a new wipe. Let the surface air dry.

## Detergent clip packs

Working with the 'Design Bugs Out' initiative, our Detergent Wipes have an integrated clip, keeping wipes neatly accessible, off the floor and nearby for whenever they are needed. The integrated clip is reinforced and angled for a firm grip and securely fits on to BP machine stands, porters chairs, bed frames, monitor stands, cabinet doors, trolleys etc.



# Order information

|                | Product                       | Units of issue | Order Code | NHSSC  |
|----------------|-------------------------------|----------------|------------|--------|
| Wipes          | Detergent Wipes               | Pack of 215    | CDW215     | VJT129 |
|                | Detergent Wipes Clip Pack     | Pack of 60     | CDCP60     | VJT488 |
| Tubs & Buckets | Detergent Wipes Tub           | Tub of 110     | CDT110     | VJT225 |
|                | Detergent Wipes Refill        | Refill of 110  | CDT110R    | VJT226 |
|                | Detergent Wipes Bucket        | Bucket of 260  | CDB260     | VJT491 |
|                | Detergent Wipes Bucket Refill | Refill of 260  | CDB260R    | VJT483 |
| Accessories    | White Wall Mounted Dispenser  | (for CDW215)   | CDWDW      | -      |
|                | Wire Wall Mounted Dispenser   | (for CDT110)   | CDT110D    | -      |